SeptoDiscs

Finishing and polishing discs

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Use: Install the mandrel at the contra-angle. Fit the disc onto the mandrel. For better results, use the four grits, successively sur the coarsest to the finest one. Product must be applied dry, with low to medium speed and with intermittent touches. It is recommended to use 10.000 rpm for the coarsest discs and 30.000 rpm for the finest.

Warnings, precautions and restrictions: Professional use only. The discs are for single use only. Its reuse may cause cross contamination. Prolonged contact of the rotary instrument with the dental tissue may cause heating compromising pulp vitality. The user will be entirely responsible for the combined use of the mandrel or the disc with similar products. To avoid an accidental swallowing of this product, it is necessary the application of a total isolation technique of the operatory field. If there is proven allergy to any of its components, the product must not be used.

Sterilization: The mandrel may be sterilized by autoclave at 126°C, 147kPa,16min. Before sterilization, proceed with the cleaning using water and enzymatic detergent.

Composition: Discs: Aluminum oxide, polyethylene terephthalate, synthetic rubber resin, metal eyelets, water-based pigments. Mandrel: Metal.

Storage: To be stored in dry and fresh place, sheltered from sunlight.

Mandrel is manufactured in compliance with specifications from ISO 1797.

Presentation:
0102000146 - SeptoDiscs - 120 assorted discs + mandrel
0102000147 - SeptoDiscs refill 1/2" - 50 assorted sandpaper discs
0102000148 - SeptoDiscs refill 1/2" - 50 discs of coarse sandpaper
0102000149 - SeptoDiscs refill 1/2" - 50 discs of medium sandpaper
0102000150 - SeptoDiscs refill 1/2" - 50 discs of fine sandpaper
0102000151 - SeptoDiscs refill 1/2" - 50 discs of x-fine sandpaper
0102000153 - SeptoDiscs refill 3/8" - 50 assorted sandpaper discs
0102000148 - SeptoDiscs refill 1/2" - 50 disques Epaisse
0102000149 - SeptoDiscs refill 1/2" - 50 disques Moyenne
0102000154 - SeptoDiscs refill 3/8" - 50 discs of medium sandpaper
0102000156 - SeptoDiscs refill 3/8" - 50 discs of fine sandpaper
0102000157 - SeptoDiscs refill 3/8" - 50 discs of x-fine sandpaper

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