

Dry socket?
Ease the pain.

Alveogyl

Dry socket dressing



Alveogyl is the ideal dry socket dressing, rapidly easing your patient's pain and convenient for you to use. Alveogyl may also be used as a post-extraction dressing.

Alveogyl is a one-step, self-eliminating treatment requiring no suturing and no special attention other than observation of the healing process. It provides a soothing effect on the alveolar tissues thus helping to rapidly alleviate the pain. Its fibrous consistency, due to the Penghawar fibres, allows for an easy filling of the socket and a good adherence to the alveola.

Features	Advantages	Benefits
Contains Penghawar fibres	Fibrous consistency	Good adherence to the alveolus during the entire healing process
Dry socket dressing	Promotes hemostasis by compression	Helps to rapidly alleviate the pain Protects from superinfection
No suturing required	No special attention other than observation of the healing process	Convenient and easy to use
Self eliminating treatment	One-step procedure	No need for recall appointments

Indications

Alveogyl is a paste used as dressing in case of dry socket or post-extraction dressing following a difficult or traumatic extraction in patients with history of dry socket.

Instructions for use

- Take a small pellet (about 0.20 g) of Aveogyl paste
- Place it gently into the prepared dental socket
- Do not suture

Product Information

10 g Jar



Septodont - Units R & S - Orchard Business Centre - St Barnabas Close - Allington - Maidstone - Kent - ME16 0JZ - UK

Tél. : +44 (0)1622 695520 - Fax : +44 (0)1622 695521

Please visit our website for more information:

www.septodont.co.uk

