fr

**COMPOSITION**

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is - **Péttiefni fyrir rötarfyllingu**

**SYMBOLS :**

	UK responsible person
	UK marking
	Unique Device Identifier

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05 94 086 36 00

UK REP

Callisto Regulatory Consulting Limited,  
 Iron Farm, 7 Grimes Gate, Diseworth, Derby,  
 DE74 2QD, United Kingdom

en

**COMPOSITION**

Hydrocortisone acetate ..... 1.0 g  
 Thymol iodide, barium sulfate, zinc oxide,  
 magnesium stearate ..... 100.0 g

**PROPERTIES**

To act as perfect root canal filling materials, pastes should be neither resorbable once hardened nor retractable over the years.

These pastes should be radiopaque and easy to insert in the canal.

ENDOMETHASONE N powder has all the various above-described qualities.

Besides, this medical device has an anti-inflammatory ancillary activity due to the presence of hydrocortisone acetate.

**PERFORMANCES**

Root canal sealer which may help reduce the post-operative pain following endodontic treatment

**CONTRE-INDICATION**

Hypersensitivity to hydrocortisone acetate or to any component of the formulation

**INDICATIONS**

Permanent obturation of permanent teeth endodontic canals with systematic introduction of gutta-percha points.

**ADVERSE EFFETS**

Risk of allergy to any of the constituents particularly in case of contact with gums.

**INSTRUCTIONS FOR USE**

The treated tooth should be isolated (using a cotton roll or a rubber dam).

After carefully shaping the canal, cleaning it with an antiseptic solution, rinsing it and drying it with paper points, prepare the canal paste extemporaneously, progressively pouring the powder into the liquid (eugenol or ENDOMETHASONE Liquid) at a ratio of 2 spoonsful of powder to 3-6 drops of liquid, until the paste has the required consistency for the selected filling technique:

- use of a paste carrier (single cone obturation)
- and/or coating Gutta-Percha points (lateral and vertical condensation)

Then insert one or several Gutta-Percha points, which have previously been fitted to the root canal shape and size.

Take X-rays to ensure that root canals have been thoroughly filled.

Close container tightly after each use.

For information:

For 14g bottle, maximum two full spoons dose estimated is 33.

For 42g bottle, maximum two full spoons dose estimated is 99.

Any unused paste should be discarded.

**INTERACTIONS**

Do not use in combination with composite materials, since eugenol-based preparations inhibit their polymerisation.

**SPECIAL WARNINGS AND PRECAUTIONS FOR USE**

As with any other root canal sealer, a canal can only be sealed if it has previously been reamed, cleaned and disinfected thoroughly.

Determine the working length of the root canal before filling so as to prevent any apical extrusion, since the paste is not resorbable.

It is recommended not to exceed 2mm from the apex

Due to electrostatics, ENDOMETHASONE N, as all powders, tends to agglomerate in the bottle. However, the capacity mentioned on the label corresponds to the weighed contents. It is therefore important to shake the bottle before use in order to loosen the powder.

The spoon is provided for use only with this product. Do not reuse once the vial has been emptied. Before use, ensure the cleanliness of the spoon in accordance with good dental practices.

Danger (contains thymol iodide).

Harmful if swallowed.

Causes severe skin burns and eye damage.

Very toxic to aquatic life with long lasting effects.

Refer to safety data sheets (available upon request).

**STORAGE**

Do not store above 25°C and protect from moisture.

**PRESENTATIONS**

- Bottle of 14 g powder + 1 spoon.
- Bottle of 42 g powder + 1 spoon.

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