| SEPTODONT<br>ARTWORK CODE | CREATION DATE | MODIFICATION DATE | DIMENSIONS<br>(MM) | COLOR(S)   | GRAPHIC DESIGNER |
|---------------------------|---------------|-------------------|--------------------|--|------------------|
| 105 93 022 02 00          | 19/07/23      | 10 août 2023      | 69,85 X 69,85 mm   |  1 | Jean             |

# Parcan N

|   |    |
|---|----|
| fr - Solution de NaOCl à 3% pour l'irrigation canalaire.....    | 2  |
| en - NaOCl 3% solution for canal irrigation.....                | 4  |
| es - Solución NaOCl al 3% para irrigación de canales .....      | 6  |
| de - NaOCl 3%ige Lösung zur Kanalspülung .....                  | 8  |
| it - Soluzione di NaOCl al 3% per l'irrigazione canalaire ..... | 10 |
| pt - Solução de NaOCl 3% para irrigação de canais .....         | 12 |
| nl - NaOCl 3% oplossing voor kanaalirrigatie .....              | 14 |
| pl - Roztwór 3-proc. NaOCl do irygacji kanałów .....            | 16 |
| el - Διάλυμα NaOCl 3% για καταιονισμό καναλιού.....             | 18 |
| bg - NaOCl 3% разтвор за напояване на канали .....              | 20 |
| cs - NaOCl 3% roztok pro proplachování kanálku.....             | 22 |
| da - NaOCl 3% opløsning til kanalvanding .....                  | 24 |
| et - NaOCl 3% lahus kanalite loputamiseks.....                  | 26 |
| hu - NaOCl 3% -os oldat a csatorna irrigálására .....           | 28 |
| lt - NaOCl 3% šķīdums kanālu apūdeņošanai .....                 | 30 |
| lv - NaOCl 3% tirpalas kanālū drēkinimui .....                  | 32 |
| ro - NaOCl 3% soluție pentru irigare canalelor.....             | 34 |
| sk - NaOCl 3% roztok na zavlažovanie kanálov .....              | 36 |
| sl - NaOCl 3% raztopina za namakanje kanalov .....              | 38 |
| sv - NaOCl 3% -lösning för avloppsbevattning .....              | 40 |

fr

**Description :**

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en

**Description:**

Parcan N 3% NaOCl is an aqueous sodium hypochlorite solution. At low concentrations dissolve mainly necrotic tissue, at higher concentrations also dissolves vital tissue. For professional use only.

**Instruction for use:**

Irrigate with Parcan N 3% NaOCl as needed for debridement during instrumentation.

1. Fill syringe with Parcan N 3% NaOCl (10 ml recommended).
2. Attach irrigation tip securely to syringe.
3. Irrigate as desired.
4. Remove and dispose of syringe and irrigating tip.

**Precautions:**

1. Read MSDS for material handling and safety information.
2. Avoid spilling on surrounding tissues.
3. Avoid spilling on colored fabrics and surfaces as Parcan N 3% NaOCl is moderately alkaline and has a whitening effect.
4. Proper treatment planning and establishment of working length is required prior to use of this product.
5. Following standard of care, isolate the tooth prior to use of this product to prevent unintended tissue contact or accidental ingestion.
6. Properly evacuate the canal during use of this product to prevent unintended tissue contact or accidental ingestion.
7. It is recommended to use irrigation tips with lateral venting (27ga or smaller) to ensure effective distribution of irrigation solution throughout the root canal and minimize the risk of apical extrusion.

**Warnings:**

1. Caution, caustic!
2. Keep out of the reach of children.
3. Avoid contact with skin and eyes.
4. Over-instrumentation prior to use of this product can result in unintended tissue contact.
5. Under-instrumentation prior to use of this product can result in tip binding and subsequent apical extrusion. Do NOT bind the tip in the root canal.
6. If tip binding occurs during the procedure, or if resistance is felt during irrigation, discontinue irrigation until the tip binding issue is resolved.
7. NEVER apply excessive pressure when irrigating NaOCl.
8. If apical extrusion of NaOCl has occurred, seek medical attention immediately.
9. If ingested, do not induce vomiting. Drink a large portion of water and provide fresh air. Seek medical attention immediately.

**Storage:**

Store in a cool, dry area away from heat and direct sunlight.

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**Inter-Med, Inc.**